Indications for use

Postoperative care of ankle surgery, sports traumatology (sprain, tear, etc.), after rehabilitation session.
Analgesic, anti-oedematous, anti-haemorrhagic effects..

Contraindications

Allergy to cold, insensitivity to cold, Raynaud's syndrome or other vasospastic diseases, arteritis, cryoglobulinemia, sickle cell disease, arteriosclerosis, diabetes, polyarthritis, disorders of skin sensitivity or hypercoagulation, peripheral vascular diseases causing ischemia or weak local blood circulation.

Risks of use and precautions for use

This medical device must be used according to the medical prescription of your doctor or according to the recommendations for use of your duly authorized health professional (pharmacist, physiotherapist, orthotist, etc.).

The manufacturer makes no warranty that use of this product will not prevent injury. It is imperative to check the completeness and integrity of the elements of the kit before use. Do not use if damaged.

Factors can increase the risk of skin burns (malnourished patients, excessive smoking, insulin-dependent diabetics, or even people with fragile skin): close medical supervision is recommended.

Cold pack should never be in direct contact with the scar or skin. Systematically use the hypoallergenic lycra supplied. Cold pack should be soft and frost free before use.

Do not heat or warm cold pack.

Respect the number of pressures indicated to inflate the pack. If tingling occurs in the limb, remove and deflate the pack immediately.

Do not pierce the cold pack, nor the manual pump and its connection pipe.

In the event of gel leakage: handle the device with gloves, do not ingest the gel, do not pour the gel into a drainage system connected to the sewers, or into a watercourse.

Do not throw the device in nature or with household waste. Disposal is done with special industrial waste, in the recycling center.

In case of direct contact of the gel with the skin, rinse immediately and abundantly with cold water.

Contact your doctor if irritation persists.

Keep out of the reach of children and people unfit to use this device.

Cleaning

Cold pack and lycra are hand washable with mild soap. Do not machine wash and never use bleach. Do not immerse in a disinfectant detergent bath.

If necessary, spray a disinfectant detergent solution (ANIOS disinfectant detergent for high surfaces).

Always place the tap in position F. Leave to air dry and lay flat.

For more information, contact Orthonov or visit the website www.orthonov.com

Warnings

This medical device is a regulated health product which follows under this regulation, the CE marking.

Before use, please read this manual carefully and follow the recommendations for use.

In the event of pain, swelling, unusual sensations, inflammatory or allergic reaction, immediately stop using the medical device and consult your authorized healthcare professional as soon as possible (pharmacist, physiotherapist, orthotist, etc.), and contact Orthonov: 01 60 13 65 73 or www.orthonov.com

In the event of a serious reaction, contact the manufacturer and the competent authority in France.

3 rue de Villebon - 91160 Saulx-les-Chartreux - France
Tél.: 01 60 13 65 73 - Fax: 01 69 15 06 37 - Courriel: contact@orthonov.com
www.orthonov.com

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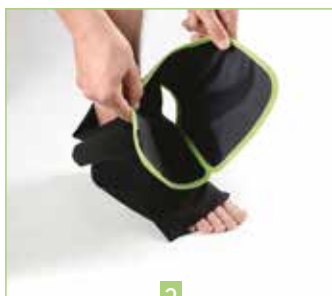
Put on the lycra clothes.
The cold pack should not never be in direct contact with the skin.

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After 8 hours of freezing (between -18 and -22 °C), take out the cold pack.

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Distribute the gel if necessary. The green border must be under the sole of the foot.

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Place the foot in the cold pack. The heel must be in the hole.

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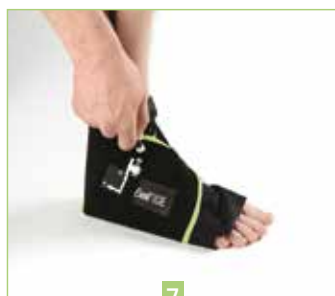
Fold down the left part of the cold pack on the foot, and take the velcro of the right side.

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
Fold down the right part of the cold pack on the left side and fix the velcro.

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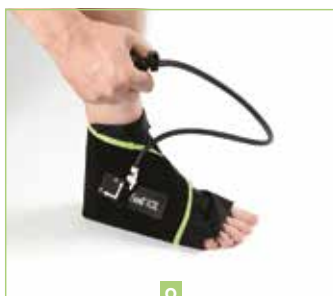Inflate (do not exceed 10 pressures). Turn the tap to F and unscrew the pump.

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After 20min, turn the tap to the O position and undo the strap. Wipe and replace the cold packs in the bag and store in the freezer..

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